

Tasectan®

Intended use

Tasectan is a medical device used to restore the physiological function of the intestinal walls. It is specifically formulated to control and reduce the symptoms associated with diarrhoea resulting from various causes, such as abdominal tension and frequent defecation. It is effective within 12 hours.

The medical device Tasectan consists of a gelatin and tannic acid complex which acts mechanically by protecting inflamed intestinal mucosa, thanks to its ability to form a protective, protein-based mucoadhesive film which complexes the mucoproteins responsible for localised inflammation and promotes their precipitation and elimination in the faeces.

The Gelatin Tannate of which the product is composed is not altered in the stomach and acts by forming a film which protects the intestinal mucosa, reducing the frequency and duration of diarrhoea episodes.

The medical device is available in capsule form for adults and as a powder recommended for paediatric use.

Nature and contents of the container

Twenty 250 mg sachets (paediatric use).

Blister pack containing fifteen 500 mg capsules.

Composition

Capsules: Gelatin Tannate, Corn Starch, Magnesium Stearate.

Powder for paediatric use: Gelatin Tannate.

Instructions for use

Capsules:

1. swallow the capsule with water.

Powder for paediatric use:

1. disperse the contents of one sachet in a glass of water. It can also be added to fruit juices or yoghurt.

Dosage

Adults: 1-2 capsules every 4-6 hours until the symptoms disappear. The dosage may be increased according to need, and in any case, always after having consulted your doctor.

Children under 3 years: 1 sachet every 6 hours until the symptoms disappear.

Children aged 3 to 14 years: 1 or 2 sachets every 6 hours until the symptoms disappear.

Warnings

Please note the following warnings:

- For paediatric use, consult your doctor prior to using this medical device.
- Do not use the medical device after the expiry date printed on the package.
- Do not swallow the capsules if the blister pack is open or damaged or if the sachet is open or damaged.
- Even though the side effects are unknown, it is recommended the product not be used during pregnancy or in the early months of breastfeeding. In any case, always consult your doctor beforehand.
- Consult your doctor should the intestinal wall irritation or diarrhoea continue.
- Store at room temperature away from sources of heat.
- Do not freeze.
- Keep the medical device out of the reach of children.

Contraindications and undesired effects

No undesired effects or contraindications are expected in association with use of this medical device.

Rev: 01 dated December 21, 2010



NOVINTETHICAL PHARMA SAGL
Via Pian Scariolo, 11
6915 Lugano - Switzerland

CE
0373